

Eclipse® Border & Eclipse® Border Oval

Super absorbent dressing with a soft silicone contact layer and border

Fact sheet

Product description

Eclipse® Border and Eclipse® Border Oval are super absorbent dressings with a self adhesive soft silicone contact layer and hypo-allergenic border. The soft silicone layer provides gentle adherence and will facilitate atraumatic removal and the hypo-allergenic border allows further adhesion.

Use

Eclipse® Border and Eclipse® Border Oval are placed face down on wound surface with beige backing uppermost allowing a minimum of a 2cm border overlap around the wound area. Eclipse® Border is self-adherent but may require additional support with a suitable bandage or tape and can be used under compression. Eclipse® Border must be changed before dressing reaches capacity but depending on wound exudate levels, may be left in place for up to 7 days.




Features

- ✓ Increased absorbency capacity
- ✓ Atraumatic and hypo-allergenic
- ✓ Breathable fluid repellent backing
- ✓ Moist wound healing environment
- ✓ Additional wicking and moisture locking central core
- ✓ Combination silicone and acrylic adhesive border
- ✓ Available in unique oval shape

Indications

Moderate to heavily exuding wounds including:

- ✓ Pressure ulcers
- ✓ Leg ulcers
- ✓ Surgical wounds
- ✓ Burns
- ✓ Graft sites
- ✓ Lymphatic legs
- ✓ Fistulas
- ✓ Fungating tumours

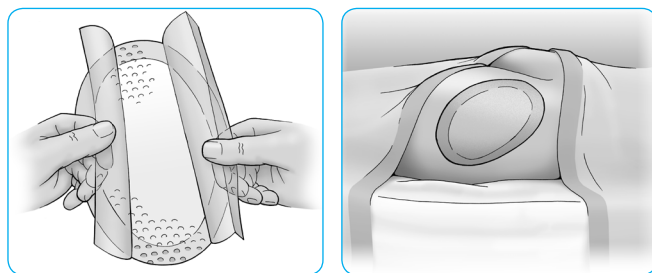

Contra-indications

Arterial bleeds and heavily bleeding wounds.
Do not cut Eclipse® Border and Eclipse® Border Oval.

Storage

Store at room temperature, out of direct sunlight and keep dry.

Ordering information for Eclipse® Border

Size	Stock code	Pack size	NHS code	PIP code
15cm x 15cm	CR4296	10	EKH061	383-6897
20cm x 30cm	CR4297	10	EKH062	383-6905

Ordering information for Eclipse® Border Oval

Size	Stock code	Pack size	NHS code	PIP code
10cm x 20cm	CR4300	10	EKH063	383-5881
15cm x 20cm	CR4299	10	EKH064	383-5899
20cm x 30cm	CR4298	10	EKH065	383-5907

Care and use symbols

- Read instructions for use
- Single use only
- Do not use if packaging is broken or damaged
- Sterilised using ethylene oxide
- CE marked medical device 0120
- Keep dry
- Store out of direct sunlight
- Do not re-sterilize

TOP TIP

A good tip is to use Eclipse® Border Oval in difficult to dress areas where corners could lift the dressing up. As there is no need to add an additional fixation, using Eclipse® Border and Eclipse® Border Oval makes dressing changes easier, saving time and money.

Advancis
Medical
www.advancis.co.uk