

Eclipse® Adherent Sacral

Fact sheet

Super absorbent sacral dressing with a soft silicone contact layer


Product description

Eclipse® Adherent Sacral is a super absorbent wound exudate management dressing for the sacral area with a Silfix® soft silicone wound contact layer.

Eclipse® Adherent Sacral will only adhere to dry skin surrounding the wound and can be easily lifted for adjustment without losing its adherent properties.

Use

Apply to wound (white silicone face down, beige side up) allowing a minimum of a 2cm border overlap around the wound area. Eclipse® Adherent Sacral must be changed before dressing reaches capacity but depending on wound exudate levels, may be left in place for up to seven days.

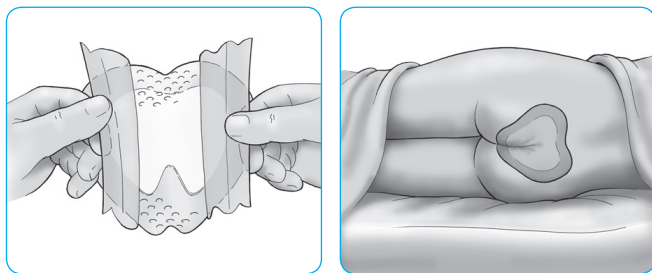

Contra-indications

Arterial bleeds and heavily bleeding wounds. Do not cut Eclipse® Adherent Sacral.

Ordering information for Eclipse® Adherent Sacral

Size	Stock code	Pack size	NHS code	PIP code
17cm x 19cm	CR3985	10	EME043	348-2213
22cm x 23cm	CR3986	10	EME044	348-2239

TOP TIP

Apply dressing in two stages; Fold in half to gain a crease down the centre, then ensure first half of dressing is sealed to skin prior to second half of dressing being exposed, to ensure good seal. This application method also reduces risk of unwanted folds and wrinkling of dressing.

Eclipse® Adherent Sacral is ideally shaped for placing on hips, underarms or breast areas as well as the sacral area.



Features

- ✓ High capacity exudate management
- ✓ Atraumatic dressing technology
- ✓ Breathable fluid repellent backing
- ✓ Moist wound healing environment
- ✓ Moisture locking system






Indications

- ✓ For all exuding wounds on the sacral area, for example pressure ulcers.

Storage

Store at room temperature, out of direct sunlight and keep dry.

Care and use symbols

	Read instructions for use		CE marked medical device
	Single use only		Keep dry
	Do not use if packaging is broken or damaged		Store out of direct sunlight
	Sterilised using ethylene oxide		Do not re-sterilize

Advancis
Medical
www.advancis.co.uk